











## TWO-HANDED (WINDLASS) **TOURNIQUET APPLICATION**

## TACTICAL FIELD CARE (TFC)

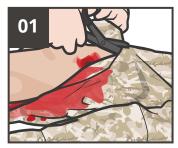

**EXPOSE** and **ASSESS** the wound.

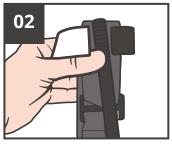

**REMOVE** tourniquet from the casualty's JFAK and/or carrying pouch.



**INSERT** wounded extremity through the loop of band (looped).

**ROUTE** band around limb, and through routing buckle.



POSITION 2-3 inches above wound directly on skin.



**PULL** self-adhering band as TIGHTLY as possible.



**FASTEN** it back on itself all the way around the limb.



TWIST the windlass rod until bleeding has stopped (complete steps 1-6 in under 1 min).



**LOCK** the windlass rod in place with the windlass clip.



**ROUTE** self-adhering band around the rod and between the clips.



**SECURE** with the windlass safety strap.



ANNOTATE time of tourniquet application.



**DOCUMENT** all findings and treatments on the DD Form1380 TCCC Casualty Card and attach to casualty.












## TWO-HANDED (RATCHET) TOURNIQUET APPLICATION

## **TACTICAL FIELD CARE (TFC)**



**EXPOSE** and **ASSESS** the wound.

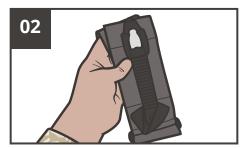

**REMOVE** tourniquet from the casualty's JFAK and/or carrying pouch.



**INSERT** wounded extremity in the loop of the tourniquet strap.




**ROUTE** band around limb, and through routing buckle.



**POSITION** 2-3 in above wound directly on skin.



**PULL** strap as TIGHTLY as possible, removing excess slack.

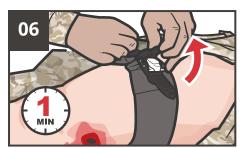

**RATCHET** device as tightly as possible until bleeding stops (complete steps 1-6 in under 1 min).



**LOCK** the ratchet on itself (it will click into place).



**WRAP** excess strap around the ratchet device and **SECURE** in place.



**DOCUMENT** the time of tourniquet application on the strap.



**DOCUMENT** all findings and treatments on the DD Form 1380 TCCC Casualty Card and attach it to the casualty.